

# **POBOVI SMART RING USER MANUAL**

## **Manualsum, simplified manuals**

### Table of Contents

- [POBOVI SMART RING USER MANUAL](#)
  - [Step 1: Activate the Ring](#)
  - [Step 2: Download the POBOVi App](#)
  - [Step 3: Connect the Ring](#)
  - [Tips 1: Ring Charging](#)
  - [Tips 2: LED Indicator](#)
  - [Tips 3: Wearing Ring](#)
  - [Tips 4: If the smart ring experiences a functional anomaly, please perform a reset operation:](#)

# **POBOVI SMART RING USER MANUAL**

POBOVI SMART RING USER MANUAL

([2BF7DPG1](#))

## **Step 1: Activate the Ring**

For first-time use , follow the “Tips 1” and place the smart ring into the charging case to charge . W hen the LED light flashes in a breathing pattern, the smart ring is successfully activated .

## Step 2: Download the POBOVi App



Search and download “POBOVi Ring” App.

## Step 3: Connect the Ring

1 Open the POBOVi App, register an account , and log in . Click “Bind Ring .”




2 Scan for devices to find the Bluetooth name “POBOVi Ring” and click to connect .



## Tips 1: Ring Charging

Please manually rotate the ring within the charging case to ensure the red U-shaped mark on the ring is near the LED indicator. The LED light will flash in a breathing pattern to indicate that the ring is charging. The charging icon (battery icon) will also appear in the app.

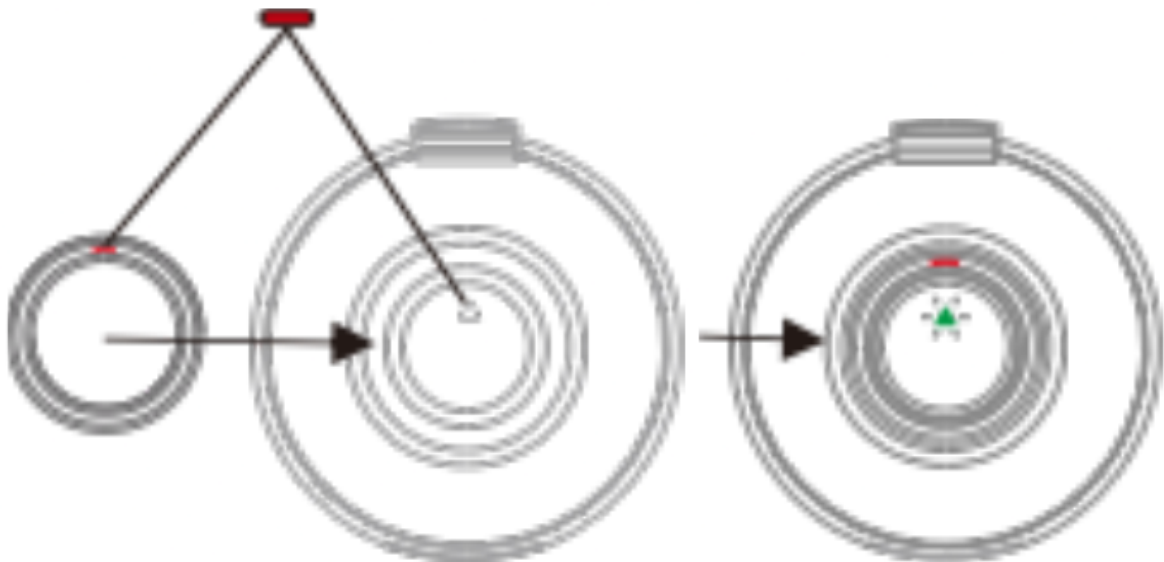

## Tips 2: LED Indicator

**Green:** Charging case is in full battery

**Blue:** Charging case is in medium battery

**Red:** Charging case is in low battery

**LED light flashes in a breathing pattern:** Ring is charging

**LED light steady on:** Charging case is currently charging .

**LED light off:** Both the charging case and the ring are fully charged .

### **Tips 3: Wearing Ring**

As shown in the figures , to ensure more accurate physiological information , please make sure the red U-shaped mark is on the back side of your finger when wearing the ring .



### **Tips 4: If the smart ring experiences a functional anomaly, please perform a reset operation:**

1. Insert the USB- C cable into the charging case to charge it .
2. Once the indicator light is steady on, place the smart ring into the charging position within 7- 10 seconds to reset it .
3. The purple light will flash for one second , indicating that the ring has been successfully reset .

**Customer Support:** pobovisupport@szelu.cn

POBOVI SMART RING USER MANUAL

[Manualsum](#)